


Surshield® Safety Winged Blood Collection Sets

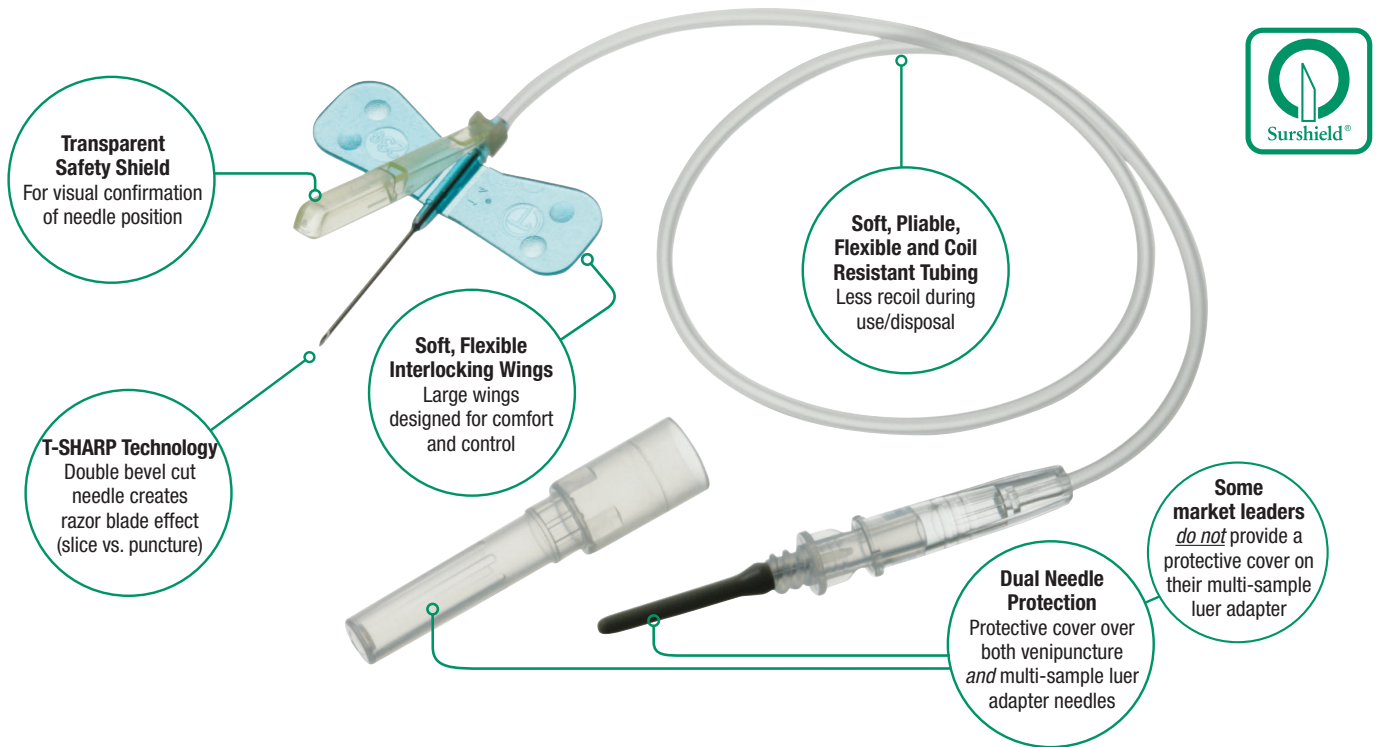


TWO MODES of One-Handed Safety Activation


Most safety winged butterflies are designed to be activated one way, and require a second hand. Surshield® provides *Two Different One-Handed Activation Options*:

THE SHIELD

- Shield cover can be turned 180° on the hinge
- Shield cover is designed to allow the user's finger to remain behind the needle point, minimizing risk of needle stick
- Shield cover is transparent for easy visual confirmation of needle placement
- Safety shield positioned top of wing - indicates bevel up

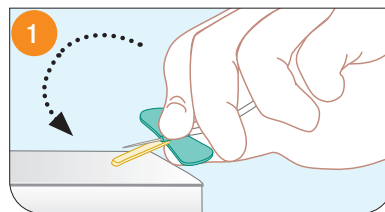

Hard surface - press the safety shield against a hard surface. An audible click signals the shield is in the locked position.

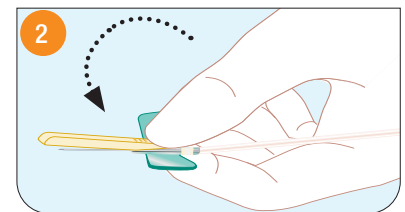

Thumb - use the thumb and forefinger to apply digital pressure to activate. An audible click signals the shield is in the locked position.

Features and Benefits*

